

Connecting the AffloVest to Bluetooth® using the app.

The AffloVest® app from Tactile Medical makes tracking therapy simple. By connecting the AffloVest controller to your phone or tablet via Bluetooth® and Wi-Fi, the app automatically records treatment data. This includes details like session length and number of treatments.

The AffloVest app can be used with the following mobile devices:

- Android 14 or newer
- Apple iOS 18 or newer operating systems — for specific device models, please refer to the following link: <https://qr1.be/2ODGQX>

1. Download the AffloVest app from the Apple iOS Store or Google Play Store on your smartphone or tablet.


- **For Android devices:** Open the Google Play Store on your device or go to play.google.com on a browser. Search for the “AffloVest App.” When you find the app, tap “Install.”
- **For Apple devices:** Open the App Store on your iPhone. Search for the “AffloVest App.” When you find the app, tap “Get.”



2. Register your account before setting up Bluetooth®. Start by filling out your name, email address, and password. Then log in with your credentials.



Fill out name, email, and password

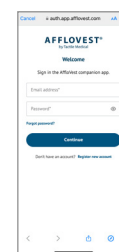

Verify your account

3. In the app, select “begin setup” to start the Bluetooth® pairing process.

4. To prepare your AffloVest for pairing with the app, navigate to the Digital Connectivity Menu on your AffloVest controller and enable the Bluetooth® feature.



Enable Bluetooth®

5. In the AffloVest app, select “search” to find your AffloVest. A 6-digit pairing code will be displayed on both the controller and the AffloVest app. Verify that the PIN characters are a match.



Verify PIN characters.

6. The Bluetooth® pairing has now been made.



7. If Wi-Fi hasn't been set up through the controller, you should continue to follow these steps to authenticate Wi-Fi via the app:

- Select “Enable Wi-Fi syncing”
- Select the preferred available network
- Enter the Wi-Fi password and click connect.
- The app will display an 8-digit PIN. Verify that it matches the PIN on the AffloVest controller.

The AffloVest is now connected to Wi-Fi and data will sync automatically.



Visit afflovest.com to learn more about our patient-preferred therapy.

AFFLOVEST®

Tactile Medical

3701 Wayzata Blvd, Suite 300
Minneapolis, MN 55416 USA

Toll Free Tel: 800.575.1900
Toll Free Fax: 866.569.1912

afflovest.com

Tactile
MEDICAL®

Bluetooth® is a registered trademark owned by Bluetooth SIG, Inc.

Tactile Medical and AffloVest are registered trademarks of Tactile Medical. ©2026 Tactile Medical. All rights reserved. 705801-001 Rev A 5/2026