

Connecting your AffloVest Controller to Wi-Fi.

The AffloVest® from Tactile Medical makes tracking your therapy simple. By connecting the AffloVest controller to Wi-Fi, AffloVest automatically records your treatment data. This includes details like session length and number of treatments.

1. Using the arrow buttons on the controller, navigate to the connectivity icon and click the play/pause button to select it.



2. Select the Wi-Fi icon, turn Wi-Fi on and select "Connect to new networks".



3. Select "Connect manually."



4. Select the preferred available network.



5. Using the on-screen keyboard and keypad, enter the Wi-Fi password and click the green checkmark.



6. You will see a screen that says “Success!
You have connected to the Wi-Fi network.”





7. Now that you’re connected to Wi-Fi, select “Authenticate my device” on the controller.



8. Tap “start authentication.” Wait a few seconds until a code appears on the controller.





9. On your computer or smart device, go to <https://auth.app.afflovest.com/activate>. Enter the code from the controller, then click “Continue.” When the confirmation screen appears, verify the code and click “Confirm.”

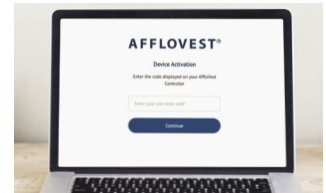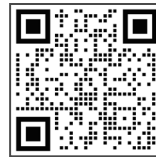

10. Register or sign-in to an AffloVest account.



11. A confirmation message will appear on both the web page and the controller when authentication is successful.

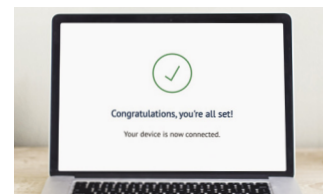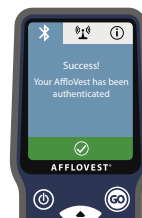

Visit afflovest.com to learn more about our patient-preferred therapy.

AFFLOVEST[®]

Tactile Medical

3701 Wayzata Blvd, Suite 300
Minneapolis, MN 55416 USA

Toll Free Tel: 800.575.1900
Toll Free Fax: 866.569.1912

afflovest.com

Tactile
MEDICAL[®]

Bluetooth[®] is a registered trademark owned by Bluetooth SIG, Inc.

Tactile Medical and AffloVest are registered trademarks of Tactile Medical. ©2026 Tactile Medical. All rights reserved. 705802-001 Rev B 7/2026